Palavit® L

- D** Gebrauchsanweisung
- GB** Instructions for use
- F** Mode d'emploi
- E** Modo de empleo
- I** Istruzioni d'impiego
- P** Instruções de uso
- NL** Gebruiksaanwijzing
- S** Bruksanvisning
- DK** Brugsanvisning
- GR** Οδηγίες χρήσεως
- RUS** Информация по использованию

Heraeus

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)
info.lab@heraeus.com
www.heraeus-kulzer.com

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GB Instructions for use Palavit® L

Resin, cold-curing, on the basis of methyl methacrylate, in the form of powder and liquid. The catalyst system contains tertiary amines Colour: yellow	Danger advice Liquid Product contains: methyl methacrylate, N,N-dimethyl-p-toluidine Highly flammable. Harmful by inhalation, in contact with skin and if swallowed. Danger of cumulative effects. Irritating to respiratory system and skin. May cause sensitisation by skin contact. Keep container in a well-ventilated place. Keep away from sources of ignition – No smoking. Take precautionary measures against static discharges. Wear suitable protective clothing and gloves. In case of fire, use sand, carbon dioxide or powdered extinguishing agent. Never use water.
Area of application Production of individual impression tray	Powder Product contains: dibenzoyl peroxide May cause sensitisation by skin contact. Keep container dry. Keep away from reducing agents, heavy metal compounds, acids and alkalis. Avoid contact with skin. Wear suitable gloves.
Processing Preparation All undercut areas which would prevent removal of the tray must be filled. Plaster model must be isolated with Aislar®	Storage advice Do not use material after expiry date. Do not store above 25 °C. Avoid direct exposure to sun. Close container immediately after use. Keep out of reach of children.
Mixing Pour liquid (approx. 8 ml) into the mixing cup and carefully add approx. 30 g powder. Mix for 1 min. Then allow to stand for 1 min. (These values apply at a room temperature of 23 °C). A slight excess of liquid will increase the working time, a slight excess of powder will decrease the working time.	Delivery units Powder: 1000 g Liquid: 500 ml
Processing Form the dough to a plate. Place the preformed plate on the plaster model and form the individual impression tray. The working time is 3 – 4 min depending on room temperatures. The periphery is trimmed during this time by removing excessive material along the aforementioned border line with a moulding knife. From excess material a handle is shaped. You can also use prefabricated handles.	Accessories Porcelain mixing jar Mixing spatula
Curing Palavit L cures in 6 to 9 min depending on room temperature.	® = Registered trademark of Heraeus Kulzer GmbH
Finishing With stones or burs. The rims can also be smoothed with sand paper.	

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® = marca registrada da Heraeus Kulzer GmbH
® = Marca registrada de la Heraeus Kulzer GmbH
® = gedeponeerd handelsmerk van de Heraeus Kulzer GmbH
® = Marchio registrato della Heraeus Kulzer GmbH