REF 15175/15176

DEU Gebrauchsinformation

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REF 15175/15176

ENG User Information

Please read the entire document before using the Silicone putty.

1. Intended purpose

Addition-crosslinked silicone putty for use in dentistry.

2. Indication

For fabrication of matrices, models for prosthetic repairs, control bites and to isolate plastic teeth in flasking.

Only for extraoral use!

2.1 Contraindication

Not for intraoral use!

3. Fields of application

3.1 General

For processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group

The target user groups for Silicone putty are:

- Dentists
- Dental technicians

The patient target group for Silicone putty includes patients receiving as part of dental treatment.

3.3 Cleaning

Silicone putty are products intended for single use. They do not require cleaning.

Silicone putty are not intended for disinfection.

4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

Fibres entrapped in the polymer are not expected to present a health hazard as long as they are processed under normal conditions of use.

5. Processing instructions

⚠ Close container immediately after removal. Do not mix up the cover and dosage spoons!

⚠ Do not wear latex gloves, latex impairs curing of the silicone modelling material. Vinyl gloves or PE-based gloves are recommended.

5.1 Dosing

Dose base and catalyst with dosage spoons at a ratio of 1:1 and spread on hand. Keep exactly to the mixing ratio to achieve correct curing times.

Overdosing the catalyst does not accelerate the curing process.

5.2 Mixing

Knead both components well for approx. 30 s until a homogenous mixture (colour) has been achieved. Process the silicone modelling material. Processing time 1,5 min.

5.3 Curing

Fully cured after approx. 4 min.

6. Storage

P234 Keep only in original container.

P402+P404 Store in a dry place. Store in a closed container.

P403+P235 Store in a well-ventilated place. Keep cool.

7. Physical data

Mixing time* 30 s; processing time* 1,5 min.; curing time* 4 min.; hardness after 24 h 85 Shore A; detailed reproduction < 1.5 µm; recovery from deformation > 99.5 %; deformation under pressure 1 %; thermal resistance 200 °C; colour silicone modelling material (base): brown, colour curing agent (catalyst): white.

*Times from start of mixing at 23°C.

8. Delivery forms

Packaging units for Silicone putty are:

- Silicone Putty
- User Information

The scope of delivery of each individual product consists of:

FINOPASTE DUO Silicone Putty
2 x 1500 g 15175
2 x 5000 g 15176

Silicone Putty are shown in the illustration:

Silicone putty are supplied without accessories.

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product

070704 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fine chemicals and chemical products not otherwise specified; other organic solvents, washing liquids and mother liquors; hazardous waste

12. Symbols explanation

12.1 Manufacturing

REF Item numbers

LOT Lot code

⌚ Shelf life

⚠ non-sterile

📦 Unit in the packaging

🏭 Manufacturer

12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

⚠

👉 Notes

📄 Important information

📖 Please read carefully before use
Keep for referral purposes

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