FINOPASTE

Silikonknetmasse - Härterpaste

REF 15180/15185/15186 15184

DEU Gebrauchsinformation

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REF 15180/15185/15186 15184

ENG User Information

Please read the entire document before using the Silicone putty.

1. Intended purpose

On C-silicone base. For fabrication of repair models, block-outs, keys (e.g. for gingival masks) and matrices.

2. Indication

Only for extraoral use!

2.1 Contraindication

Not for intraoral use!

3. Fields of application

3.1 General

For processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group

The target user groups for Silicone putty are:

- Dentists
- Dental technicians

The patient target group for Silicone putty includes patients receiving as part of dental treatment.

3.3 Cleaning

Silicone putty are products intended for single use. They do not require cleaning.

Silicone putty are not intended for disinfection.

4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

Silicone putty:

No particular hazards worth mentioning.

Hardener:

Attention

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H332 Harmful if inhaled.

H373 May cause damage to organs through prolonged or repeated exposure.

4.1 Prevention

P202 Do not handle until all safety precautions have been read and understood.

P260 Do not breathe dust/fume/gas/mist/ vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

4.2 Response

P302+P352 IF ON SKIN: Wash with plenty of water.

P332+P313 If skin irritation occurs: Get medical advice/attention.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

P314 Get medical advice/attention if you feel unwell.

5. Processing instructions

5.1 Dosing

Dose a circular measuring spoon of silicone modelling material and wipe off excess.

Spread the dosed quantities on the hand.

For each measuring spoon of silicone putty used, press in the edge of the tray once.

Dose a strand of curing paste per circular diameter.

Close container tightly immediately after use.

5.2 Mixing

Fold the silicone modelling material and knead both components well for 30-45 s until a homogeneous mixture (colour) has been achieved. Process the silicone modelling material.

The stated processing time is based on measurements at 23°C/ 73°F room temperature and 50% relative humidity.

5.3 Curing

Curing time at least 5-7 min. at room temperature. The curing time is accelerated by more intensive kneading, overdosing of the curing agent or high outside temperatures.

Low temperatures or underdosing of the curing agent slow down the curing process.

6. Storage

Store tightly closed at a temperature of 15-23°C/ 59-73 °F and 50% relative humidity.

Avoid heat and sunlight!

Shelf life: see label

7. Physical data

Mixing time 30-45 s; processing time 90 s; setting time at room temperature 5-7 min.; hardness: 66 Shore A; mixing ratio putty: activator: 11.0 g: 0.3 g; max. over- and underdosing: 20%; Colour silicone putty (base): light grey, colour curing paste (catalyst): blue.

7.1 Composition

Contains: Alkyl-silicates.

8. Delivery forms

Packaging units for Silicone putty are:

- Silicone Putty - Hardener
- User Information

The scope of delivery of each individual product consists of:

FINOPASTE

Silicone putty	1400 g	15180
	4500 g	15185
	15000 g	15186

Hardener	40 ml	15184
----------	-------	-------

Silicone Putty - Hardener are shown in the illustration:

Silicone putty are supplied without accessories.

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product

070704 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fine chemicals and chemical products not otherwise specified; other organic solvents, washing liquids and mother liquors; hazardous waste

12. Symbols explanation

12.1 Manufacturing

REF Item numbers

LOT Lot code

Shelf life

non-sterile

Unit in the packaging

Manufacturer

12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

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