REF 15191/15192/15193/15194 15184

DEU Gebrauchsinformation

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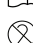
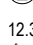
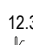
REF 15191/15192/15193/15194 15184

ENG User Information

Please read the entire document before using the Silicone putty.

1. Intended purpose

Condensation-crosslinked silicone putty for use in dentistry.

2. Indication

For fabrication of matrices, models for prosthetic repairs, control bites and to isolate plastic teeth in flasking.

Only for extraoral use!

2.1 Contraindication

Not for intraoral use!

3. Fields of application

3.1 General

For processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

3.2 Target group

The target user groups for Silicone putty are:

- Dentists
- Dental technicians

The patient target group for Silicone putty includes patients receiving as part of dental treatment.

3.3 Cleaning

Silicone putty are products intended for single use. They do not require cleaning.

Silicone putty are not intended for disinfection.

4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

Silicone putty:

No particular hazards worth mentioning.

Hardener:

Attention

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H332 Harmful if inhaled.

H373 May cause damage to organs through prolonged or repeated exposure.

4.1 Prevention

P202 Do not handle until all safety precautions have been read and understood.

P260 Do not breathe dust/fume/gas/mist/ vapours/spray.

P280 Wear protective gloves/protective clothing/ eye protection/face protection.

4.2 Response

P302+P352 IF ON SKIN: Wash with plenty of water.

P332+P313 If skin irritation occurs: Get medical advice/attention.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P337+P313 If eye irritation persists: Get medical advice/attention.

P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

P314 Get medical advice/attention if you feel unwell.

5. Processing instructions

Close container immediately after removal. Do not mix up the cover and dosage spoons!

5.1 Dosing

Recommendation: one 4.0 cm long strand of hardening paste per dosage spoon of FINOPASTE silicone putty. Over- or underdosing of the hardening paste accelerates or slows down the setting process.

5.2 Mixing

Spread the dosed quantities on the hand. Knead both components well for approx. 40 s until a homogenous mixture (colour) has been achieved. Process the silicone modelling material.

5.3 Curing

Complete curing after approx. 2 minutes.

6. Storage

Store tightly closed at a temperature of 15-23°C/ 59-73 °F and 50% relative humidity.

Avoid heat and sunlight!

Shelf life: see label

7. Physical data

Mixing time* 40 s; processing time* 2 min.; curing time* 7 min.; hardness after 24 h 80 or 90 Shore A; thermal resistance > 130 °C; colour silicone modelling material (base) S80: light grey, S90: blue, colour curing paste (catalyst): blue.

*Times from start of mixing at 23°C.

7.1 Composition

Hardener:

Contains: Alkyl-silicates.

8. Delivery forms

Packaging units for Silicone putty are:

- Silicone Putty - Hardener
- User Information

The scope of delivery of each individual product consists of:

Product	Weight	Item No.
FINOPASTE S80 silicone putty	1500 g	15191
	10 kg	19193
Trial pack	150 g	15191P
FINOPASTE S90 silicone putty	1500 g	15192
	10 kg	15194
Trial pack	150 g	15192P
Hardener	40 ml	15184

Silicone Putty - Hardener are shown in the illustration:

Silicone putty are supplied without accessories.

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product

070704 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of fine chemicals and chemical products not otherwise specified; other organic solvents, washing liquids and mother liquors; hazardous waste

12. Symbols explanation

12.1 Manufacturing

Item numbers

Lot code

Shelf life

non-sterile

Unit in the packaging

Manufacturer

12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

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