3M ESPE**Rocatec™****Adhesion System for Metal, Ceramic, and Plastic Materials Not Requiring Retention****(D) Retentionsfreies Haftverbundsystem für Metall, Keramik, Kunststoff****(F) Système adhésif sans rétentions mécaniques, pour le métal, la céramique et les matières synthétiques****(I) Sistema adesivo privo di ritenzione per metallo, ceramica, materiale plastico****(E) Sistema «compound» de adhesión sin retenciones mecánicas para metal, porcelana y plástico****(P) Sistema de união adesiva sem retenção mecânica para metal, cerâmica e material sintético****(NL) Retentievrij hechtsysteem voor metaal, keramiek, kunststof****(GR) Σύστημα συγκόλλησης χωρίς δημιουργία οπιμών συγκράτησης για μετάλλο, πορεάλν, ακρυλικό****(S) Retentionsfritt adhesivsystem för metall, keramik och plast****(FIN) Kiinnitysjärjestelmä non-retenttiivisille metalli-, keramiikkaja-, muovipinnaille****(DK) Retentionsfrit bindingssystem til metal, keramik, kunststof****(N) Adhesjonsystem for metall, keramikk, akrylbaserete materialer uten bruk av mekanisk retasjon****Instructions for Use
Gebrauchsinformation
Notice d'utilisation
Informazioni per l'uso
Información del empleo
Instruções de uso****Gebruiks informatie
Πληροφορίες χρήσεως
Bruksanvisning
Käyttöohjeet
Brugervejledning
Bruksveileiding**

440073176101

ENGLISH**Product Description**

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ESPE Sil 5 min (Trocknenzeit)

Zeiten

Rocatec Pre 10 sec/2,8 bar (blasting time for each veneer unit/ blasting pressure)

Rocatec Plus 13 sec/2,8 bar (blasting time for each veneer unit/ blasting pressure)

Rocatec Soft 13 sec/2,8 bar (blasting time for each veneer unit/ blasting pressure)

ESPE Sil 5 min (drying time)

Notas

Proper adhesive bonding is ensured only if the genuine ESPE coating device is used! It is essential to use the blasting material with the designated blasting tank (see colour coding on the package and the blasting tank). Errors in the assignment of blasting material and tank will lead to bonding failure. Contaminated or spent blasting material must not be used or re-used!

All steps of the procedure, from preliminary blasting to opäquer application, must be conducted carefully, in the stated sequence, and without extended interruptions to provide for optimal adhesion.

Inadequate mixing of the opäquer impairs bonding. Any contamination of the veneer surfaces, e.g., by fat, must be avoided!

Serious bonding defects may result from overheating of alloy melts or the exclusive use of metal which has been melted repeatedly!

Faulty frame design and inappropriate design of the free incisal edge may lead to defects that are not causally related to the bonding procedure.

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